
INSVET Pet Scan RT 100 V8

Contenido

1.	Ayuda en Español	2
1.1.	Entorno INSVETPetScanRT100V8	2
1.2.	Aspectos generales básicos.....	4
1.3.	Microchips identificables	5
2.	Ajuda en Português	6
2.1.	Ambiente INSVETPetScanRT100V8	6
2.2.	Visão geral	8
2.3.	Microchips identificados	9
3.	English Help	10
3.1.	INSVETPetScanRT100V8 Enviroment	10
3.2.	Overview	12
3.3.	Identified Microchips	13

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1.1. Entorno INSVETPetScanRT100V8

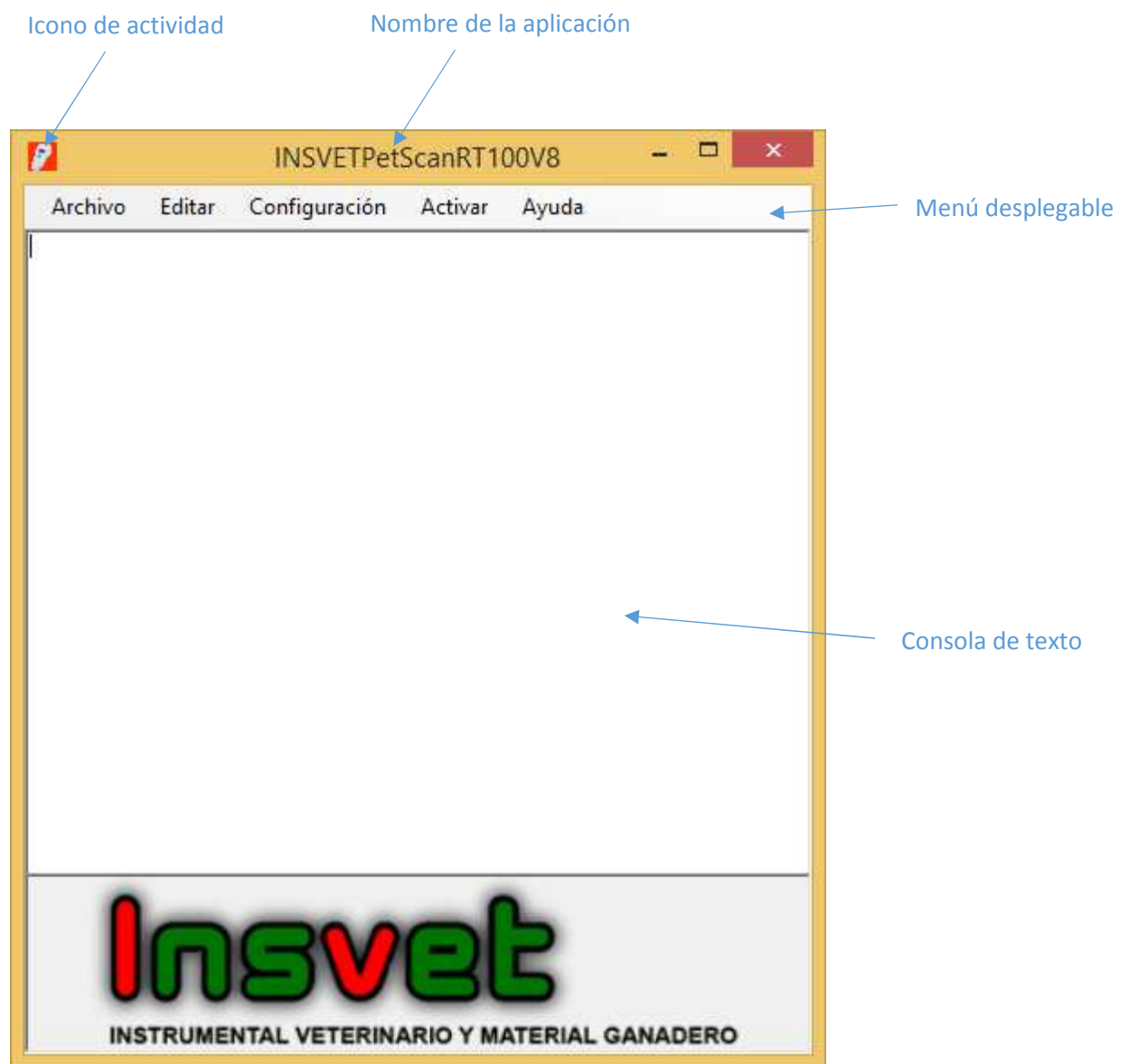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
3. English Help

3.1. INSVETPetScanRT100V8 Enviroment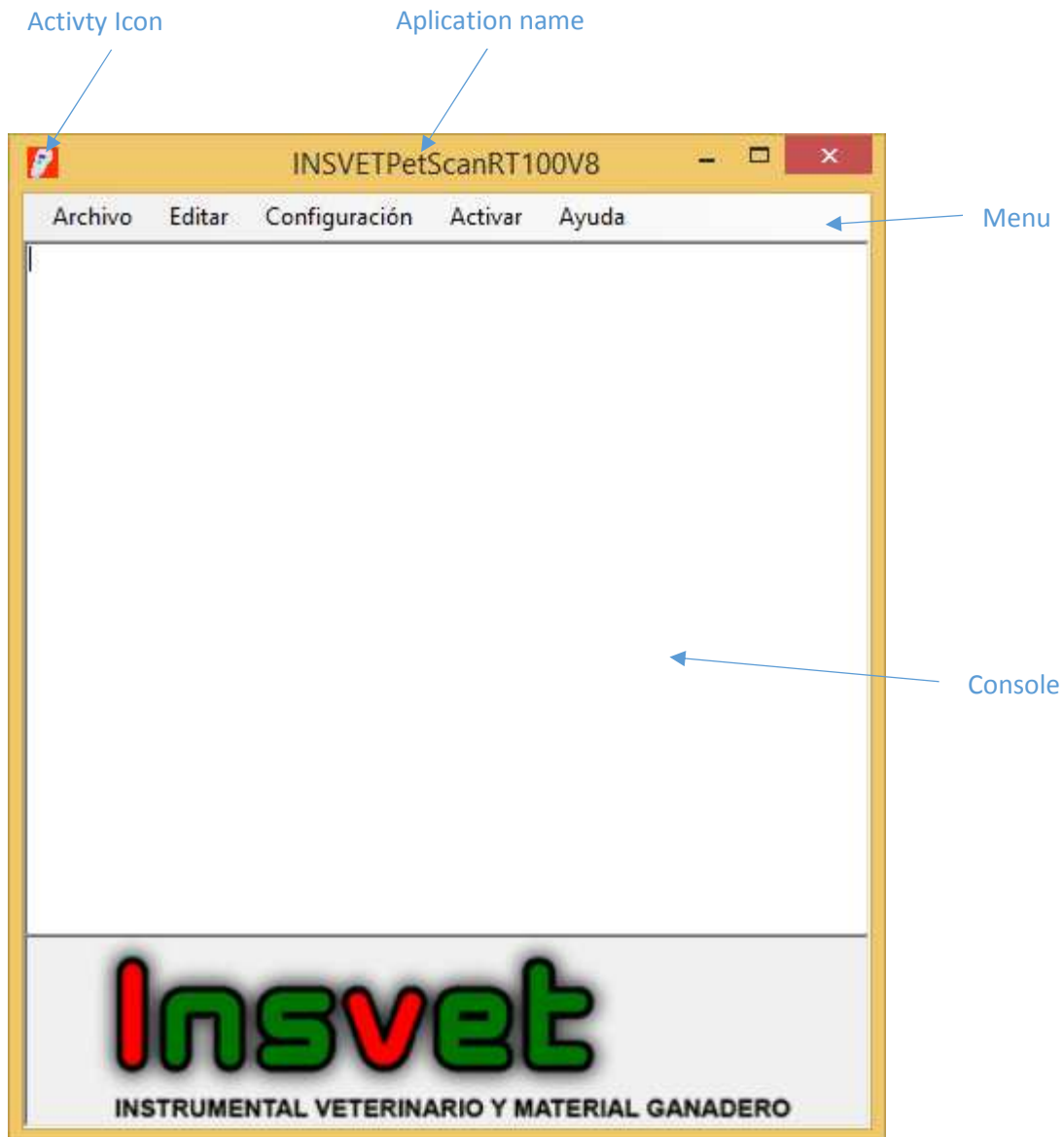

The main application menu has the following items:

Archive:

Open File → opens a text file (txt or rtf).

Save File → saves a text file (txt or rtf). Admits saving files in RTF format, i.e.: supports pasting images.

Export File:

Saves a Word Doc file (plain text)

Saves an Excel csv file (comma separated file)

Print: Prints directly the values are loaded into the Console.

Edit:**Cut, Copy and Paste****Configuration:**

Port → Select the port COM through which the application will communicate. The application automatically selects the port only if there is no active bluetooth port on the computer and connect the scanner via the USB cable.

***Note:** When you connect the computer via bluetooth port check if the blue light of the reader remains stable without blinking, otherwise the application will throw an error and stop the app activity until the user selects a port manually and/or press the On button (this is located on the menu and acts as a switch). Remember to always turn on the bluetooth of the scanner before launching the application.*

Language → sets the default language of the application.

Start with Windows → sets whether the application will start each time you open windows. The first time you install the program it is configured to start at boot windows and is available from the first time you load by default.

***Note:** When the program is configured to start by default with windows, turning on computer and start Windows if your last connection was through a bluetooth port the application will throw an error and activity dentendrá app if the reader was not on previously .*

Start → is a part of the Main Menu. You can switch between Start/Stop values. By clicking on it, the application state changes, it is easily identifiable through the notification icon in the taskbar:

- Red = Application detained (switch menu displays the Start text).
- Green = Application running (switch menu will display the Stop text).

Help

Help → Displays this document.

Version → Information about the program version and data about the operating system of the machine on which it is installed as well as its architecture (32/64bit).


3.2. Overview

The first time you start the application must select the connection port for the scanner. Previously the bluetooth and the scanner must be paired, otherwise the scanner must be connected to your computer using the USB cable.

For selecting a new port you must go to: Setup menú → Ports → and then select a COM port from the drop down menu. Each time you display the menu, the program will detect whether there are new connection ports (i.e: pairing a bluetooth scanner or connecting the USB)


The application always displays an icon in the taskbar. The icon can indicate the state of the app like its counterpart in the form (color code: green = active, red = stopped). In addition, the application form also allows to leave it unseen by unchecking the View Application. Another options are: Start or Stop the application and close it.

Note 1: The first time you install the app it will remain visible until you deselect the item Show Form icon on the taskbar.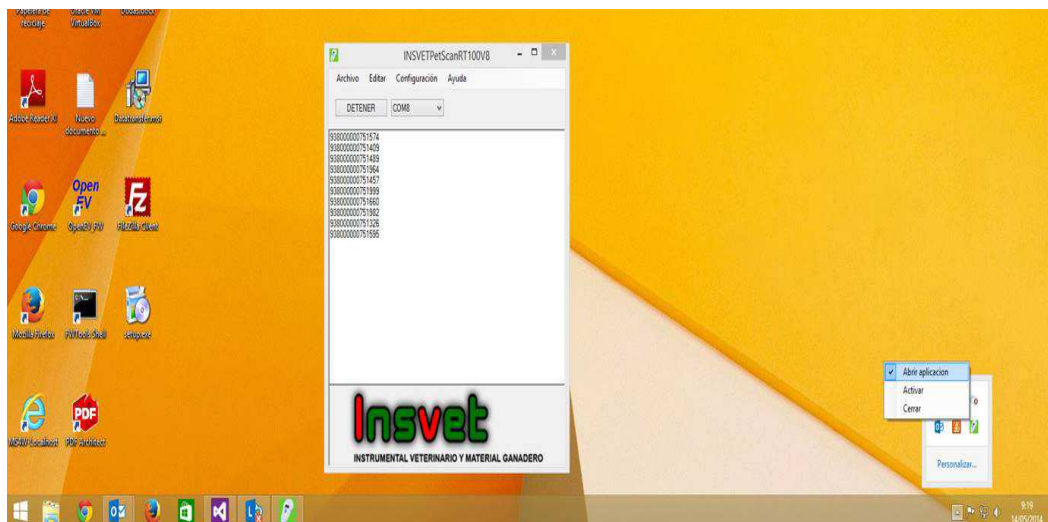

Note 2: Once the application is installed and running, each time you perform a scan of a microchip will enter your numbers in the active window. You can work with the window of the app or directly on Excel, Word or any application that allows you to type texts.

View of the notification icon in the taskbar: active Application vs. arrested Application (from far left position to right).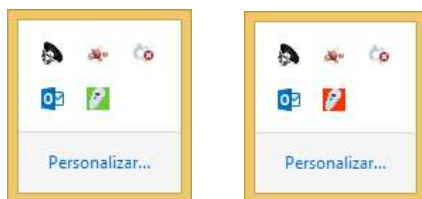

Note 3: *The application stores the last user-defined settings for both the language and the port you configured for the scanner. Remember that if you decided to work using the bluetooth port, you must pair your computer and the scanner (the use of codes is not necessary). The bluetooth port configuration may take several minutes.*

3.3. Identified Microchips

ISO FDX B (15 dígitos) e ISO FDX A (10 dígitos)

HDX y EM4102